



P.O. Box: 48577, **DUBAI**, U.A.E.

Tel: 04 267 8866, Fax: 04 267 8855

P.O. Box: 457, **SHARJAH**, U.A.E.

Tel: 06 565 8866, Fax: 06 565 6699

E-mail: [info@zulekha-hospitals.com](mailto:info@zulekha-hospitals.com) • Website: [www.zulekha-hospitals.com](http://www.zulekha-hospitals.com)

ص.ب. ٤٨٥٧٧، دبي، إ.ع.م.

هاتف: ٠٤ ٢٦٧ ٨٨٦٦ - فاكس: ٠٤ ٢٦٧ ٨٨٥٥

ص.ب. ٤٥٧، الشارقة، إ.ع.م.

هاتف: ٠٦ ٥٦٥ ٨٨٦٦ - فاكس: ٠٦ ٥٦٥ ٦٦٩٩



## **SUCTION REGULATORS FOR CENTRALIZED SUCTION SYSTEM** **(ZH DXB ONLY)**

- Use the GREEN/RED PUSH Switch for switching ON and OFF the controller
- The level of the vacuum required could be adjusted using the vacuum control knob
- The unit could be tested by pinching the patient tube connection and adjusting the vacuum control knob to the maximum position, then ensure that the gauge reads the maximum value (Adult regulator > 800 mbar & Neonatal regulator > 150 mbar)

ON / OFF PUSH  
Switch  
RED - ON  
GREEN - OFF

Additional  
overflow  
protection jar

