



P.O. Box: 48577, **DUBAI**, U.A.E.
Tel: 04 267 8866, Fax: 04 267 8855
P.O. Box: 457, **SHARJAH**, U.A.E.
Tel: 06 565 8866, Fax: 06 565 6699

E-mail: info@zulekha-hospitals.com • Website: www.zulekha-hospitals.com

صندوق بريد: 48577، دبي، الإمارات
هاتف: 04 267 8866 - فاكس: 04 267 8855
صندوق بريد: 457، الشارقة، الإمارات
هاتف: 06 565 8866 - فاكس: 06 565 6699



USAGE INSTRUCTIONS, CARE AND MAINTENANCE OF BRAUN THERMOSCAN

- To achieve accurate readings, make sure a new clean probe cover is in place before each measurement
- When the probe cover is in place, the unit turns on automatically. Wait for the ready signal to beep
- Fit the probe snugly into the ear canal, then push the start button, The unit will beep once. The green ExacTemp light above the start button will start flashing indicating correct probe placement and an accurate reading being taken.
- A long beep and solid green ExacTemp light signals the end of the measurement.
- If the probe is not placed in stable position during measurement, the device beeps and the green ExacTemp light goes out and the unit will give a POS (position error).
- If the patient has hearing aid, remove the same and wait for 20 minutes before taking the measurement.
- Never use the thermoscan without the probe covers installed.
- Damaged, perforated or soiled probe covers should not be used.
- Ensure that the probe window is clean dry and undamaged before each use.
- If the probe window is soiled with ear wax (possible only if the probe covers are not used), dust or finger prints, gently wipe its surface with a cotton swab slightly moistened with alcohol and immediately wipe dry with a clean cotton swab. After cleaning allow at least 5 minutes drying time before taking measurements.
- If the probe window is damaged, send the thermometer to biomedical department for confirmation. Once the damage is confirmed return the same to the store department.